

FibriJet® Procedure Kits Assembly Instructions

FibriJet®

Biomaterials Applicators

The FibriJet® Ratio Applicator has EVOLVED!

SA-4400
Procedure Kit
shown with
SA-3615 Applicator Tip
(sold separately).



1

Remove Applicator Tip from sterile package, place on sterile field.



2

Remove Applicator Kit or Assembly from sterile pouch and place on sterile field.



3

Fill 12cc syringe with desired liquid.



4

Fill 1cc syringe with desired liquid until plunger height is even with 12cc syringe.



5

Rotate plunger link from 12cc syringe and attach to 1cc syringe.



6

Attach Applicator Tip to distal luer fittings; rotate Twist-2-Lok™ Connectors clockwise to secure.



7

Applicator is ready to use



www.fibrijet.com

1270 Eagan Industrial Road • St. Paul, MN 55121 USA
PH: 651-452-1977 • TOLL FREE: 800-624-5662

©2009 Micromedics, Inc. All rights reserved.
FibriJet is a registered trademark of Micromedics, Inc.
M-FJ-0114/B