

Silverstein



MicroWick™

Drug Delivery System

OTOLOGY




- Delivers treatment fluids to the middle ear
- Patients can continue self-treatment at home
- Excellent clinical results¹
- Simple office procedure
- Kit includes:
 - Two pre-cut absorbent wicks
 - One tympanostomy tube

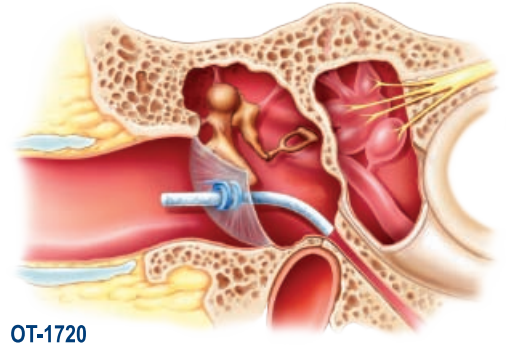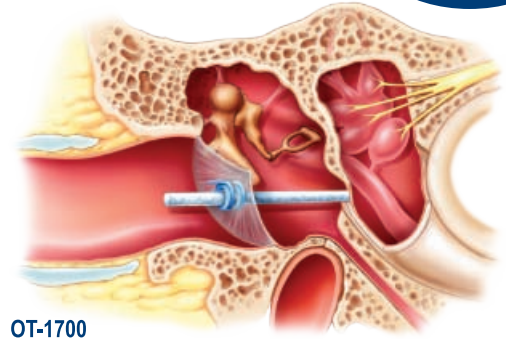

In this procedure, which can be performed in the office, the physician places the wick through a special tympanic membrane vent tube and directs it to the round window membrane or orifice of the eustachian tube. The wick inserts easily before hydration and locks securely into place after hydration. With hydration, medication applied to the MicroWick is transported directly to the round window membrane and to the fluids of the inner ear or the orifice of the eustachian tube.

Item Number	Description	Dimensions	Sale Unit / Box <i>Sterile kits, single use</i>
OT-1700	MicroWick	9mm x 1mm	1
OT-1720	MicroWick ET	19mm x 1mm	1

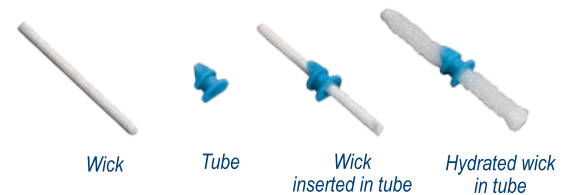

Literature and Support Materials:

¹Silverstein, Herbert, MD, FACS, *Use of a New Device, the MicroWick, to Deliver Medications to the Inner Ear*, ENT Journal, August 1999, Vol. 78 No. 8

Silverstein, H., Light, J.P., Jackson, L., et al., *Direct Application of Dexamethasone for the Treatment of Chronic Eustachian Tube Dysfunction*, ENT Journal, January 2003, Vol. 82 No. 1

Silverstein, H., Lewis W.B., Jackson, L., et al., *Changing Trends in the Surgical Treatment of Meniere's Disease: Results of a 10-year Survey*, ENT Journal 2002, Vol. 82 No. 3

Silverstein, Herbert, MD, FACS, *Clinical Use of MicroWick™ Products – CD ROM*.

Hill III, S., Digges, N., Silverstein, H., *Long-term Follow-up after Gentamicin Application Via the Silverstein MicroWick in the Treatment of Meniere's Disease*, ENT Journal, August 2006, Vol. 85 No. 8

Call **1.800.624.5662** to order today!

Micromedics, Inc. | St. Paul, MN USA | local p: 651.452.1977 f: 651.452.1787 | toll free p: 1.800.624.5662 f: 1.888.504.0606



Micromedics

www.micromedics.com

CAUTION: Federal (USA) law restricts these devices to sale by or on the order of a physician. Indications, contraindications, warnings and instructions for use can be found in the product labeling supplied with each device.
©2010 Micromedics, Inc. MicroWick is a trademark of Micromedics, Inc. All rights reserved. M-ENT-0012/D MAY10